


FINALLY, AN ALTERNATIVE TO SALINE IRRIGATION

IRRISEPT® O.R.

0.05% Chlorhexidine Gluconate (CHG) in Water for Irrigation, USP
An FDA-Cleared Cleansing and Debridement System

IRRISEPT O.R. IS EASY TO USE:

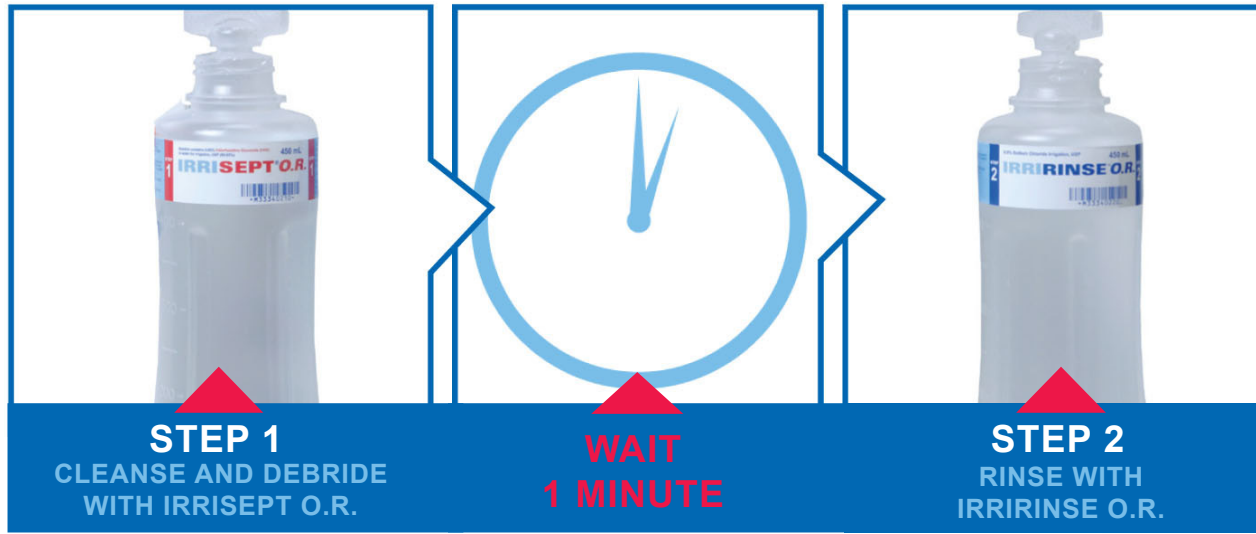

NOTE: Remove applicator used in Step 1 and reuse for Step 2.




Irrisept® O.R.
with IrriProbe®

IRRIGATION APPLICATORS:


SplatterGuard®

The SplatterGuard facilitates splatter protection and decreases aerosolized contamination during wound cleansing and debridement procedures.

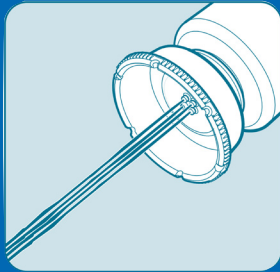

LT SplatterGuard®

The LT SplatterGuard facilitates splatter protection and decreases aerosolized contamination during wound cleansing and debridement of abscesses.

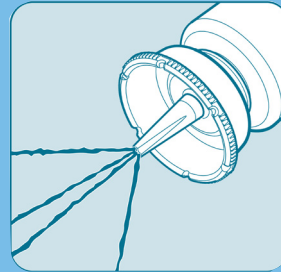

IrriProbe®

The IrriProbe facilitates deep tissue wound cleansing and debridement in an operating room setting.



EXAMPLES OF IRRISEPT O.R. USE INCLUDE:

- ▶ **Surgical Procedures** (as a final rinse before closure)
 - Orthopedic Surgery
 - General Surgery
 - Plastic & Reconstructive Surgery
 - Cardiothoracic Surgery
 - Neurologic Surgery
- ▶ **Surgical Site Infections (SSI)**
- ▶ **Skin & Soft Tissue Infections (SSTI)**
- ▶ **Delayed Closure**
- ▶ **Wound Dehiscence**
- ▶ **Abscesses**
- ▶ **Deep Traumatic Wounds**
- ▶ **Pilonidal Cysts**
- ▶ **Burns**

Irrisept is indicated for use on wounds. Contraindicated for patients with CHG allergy. Keep away from eyes and ear canals; if there is contact with these areas, rinse out promptly and thoroughly with water or normal saline.

HOW IT WORKS:

Through manual bottle compression, the patented Irrisept O.R. system delivers 450mL of solution in less than 30 seconds to effectively loosen and remove wound debris and bacteria without harming underlying tissues.

ORDER INFORMATION:

- Each Irrisept O.R. system contains:
- (1) Irrisept O.R.— 450 mL bottle with 0.05% Chlorhexidine Gluconate (CHG) in water for irrigation, USP (99.95%)
 - (1) Irririse O.R.— 450 mL bottle with 0.9% Sodium Chloride, USP
 - (1) SplatterGuard
 - (1) LT SplatterGuard
 - (1) IrriProbe
- Catalog #: Irrisept 403**
Order Unit: Case / 6 eaches



Order #: _____

InService Date(s): _____ Time(s): _____

Irrisept can be found in: _____

Local Representative: _____